TEKNİK VERİLER

Entra B 20 | 10 miniBTE T R

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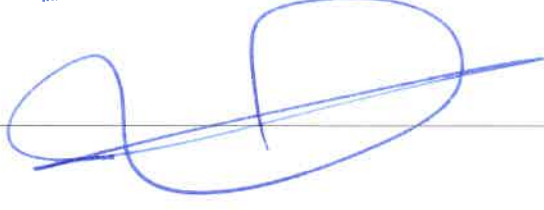
 SBO Hearing A/S
Kongebakken 9
DK-2765 Smørum
Danimarka

www.bernafon.com

Bernafon is part of the Demant Group.

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TECHNICAL DATA

Entra B 20 | 10 miniBTE T R

Bernafon Entra B is a hearing instrument with Hybrid Technology™. The miniBTE T R is an easy-to-use, rechargeable hearing instrument with a lithium-ion battery to provide power for a full day of use, including direct audio streaming. It is a behind-the-ear hearing instrument designed for users with

slight to severe hearing losses. It includes Bluetooth® Low Energy and NFMI technology, a telecoil, and single push-button for volume and program changes. The miniBTE T R is available with the miniFit thin tube system, which includes a variety of domes and custom moulds.

Hook

MNB T R

Thin tube 1.3 mm

MNB T R

Thin tube 0.9 mm

MNB T R

Technical features

- Direct audio streaming¹
- Hands-free communication²
- Bluetooth Low Energy technology
- NFMI (Near-Field Magnetic Induction)
- Single push-button
- Telecoil
- Thin tube thin tube
- Hydrophobic coating
- LED visual indicator

Accessories

- Bernafon App
- RC-A (remote control)
- TV-A (TV adapter)
- SoundClip-A
- Charger miniBTE T R
- Charger Plus miniBTE T R

General Features

- Digital Programmable
- Automatic or Manual Volume Control
- Maximum Output Control System
- MPO-Maximum Power Output
- GC-Gain Control
- AGC-Automatic Gain Control
- Noise Reduction
- Feedback Management
- Dual Microphone
- FM Compatible (with Telecoil)
- 4 Programs
- 48 channels

For information on compatibility, please visit www.bernafon.com/compatibility

Operating and charging conditions

Temperature: +5°C to +40°C (41°F to 104°F)
Humidity: 5% to 93% relative humidity,
non-condensing
Atmospheric pressure: 700 hPa to 1060 hPa

Transportation and storage conditions

Temperature and humidity shall not exceed the
mentioned limits for extended periods during
transportation and storage:

Transport

Temperature: -20°C to +60°C (-4°F to 140°F)
Humidity: 5% to 93% relative humidity,
non-condensing
Atmospheric pressure: 700 hPa to 1060 hPa

Storage

Temperature: -20°C to +30°C (-4°F to 86°F)
Humidity: 5% to 93% relative humidity,
non-condensing
Atmospheric pressure: 700 hPa to 1060 hPa

1) From iPhone, iPad, Mac and select Android devices

2) Hands-free communication is available on select devices

WARNING: No modification of this equipment is allowed.

Apple, the Apple logo, iPhone, iPad, Mac and the Mac logo are trademarks of Apple Inc., registered in the U.S. and other countries. Use of the Made for Apple badge means that an accessory has been designed to connect specifically to the Apple product(s) identified in the badge, and has been certified by the developer to meet Apple performance standards. Apple is not responsible for the operation of this device or its compliance with safety and regulatory standards. Android™ is a trademark of Google LLC.

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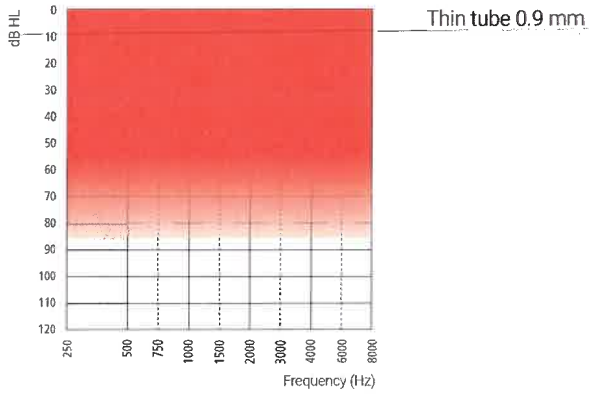
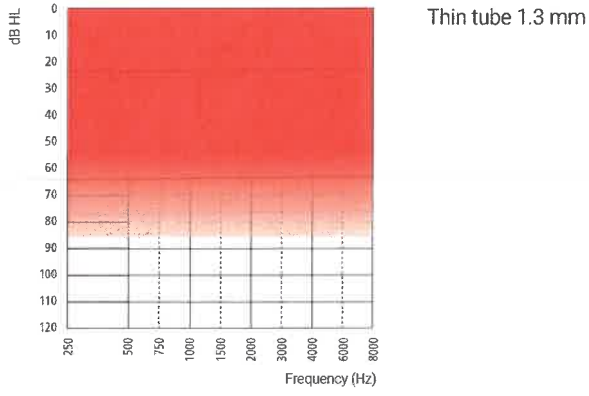
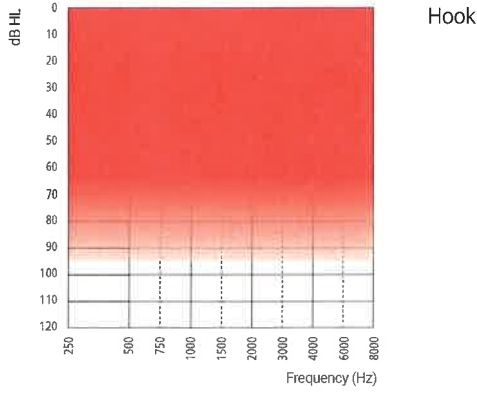
[Handwritten signature]

Feature overview

	Entra B 20	Entra B 10
Hybrid Technology™		
Hybrid Sound Processing™	•	•
Frequency bandwidth	8 kHz	8 kHz
Speech Balancer	•	•
Hybrid Noise Management™	•	•
Smart Noise Reduction	3 options	2 options
Smart Directionality	4 options	3 options
Hybrid Feedback Canceller™	•	•
Speech		
Low Frequency Enhancer	•	•
Frequency Composition ^{ext}	•	•
Comfort		
Transient Noise Reduction	2 options	—
Wind Noise Manager	•	•
Soft Noise Manager	•	•
Directionality controls		
Dynamic	•	—
Adaptive Full Directionality	•	•
Fixed Directionality	•	•
Fixed Omni	•	•
Individualisation		
Personalisation	•	•
Fitting bands	14	12
Program options/memories	10/4	8/4
Music Experience	•	—
Binaural coordination: VC, program change	•	•
Automatic Adaptation Manager	•	•
Transition	•	•
Data Logging	•	•
Tinnitus SoundSupport	•	•
CROS compatibility	•	•

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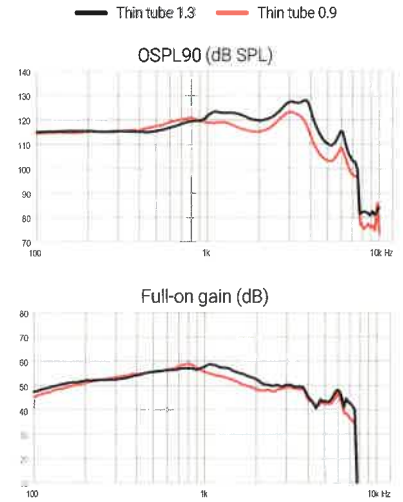
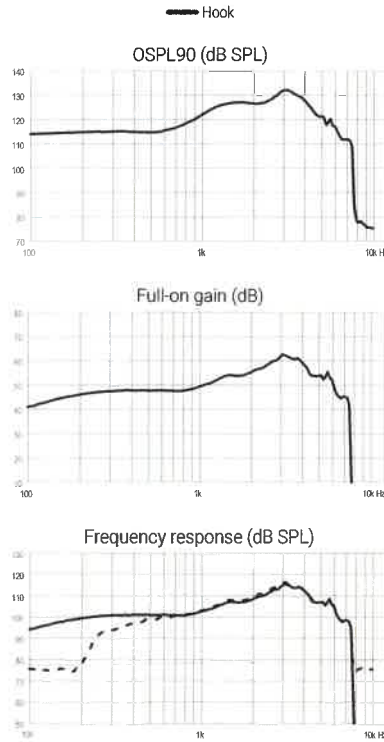
- Full-on gain is measured with the gain control of the hearing instruments set to its full-on position minus 20 dB and with an input SPL of 70 dB. This is to obtain a gain response equal to the full-on gain response from e.g. IEC 60118-0+A1:1994 but without influence of feedback.
- Measurement is done in quiescent mode. Expected use time for rechargeable battery depends on use pattern, active feature set, hearing loss, sound environment, battery age and use of wireless accessories.

Entra B 20 | 10 miniBTE T R

2CC Coupler

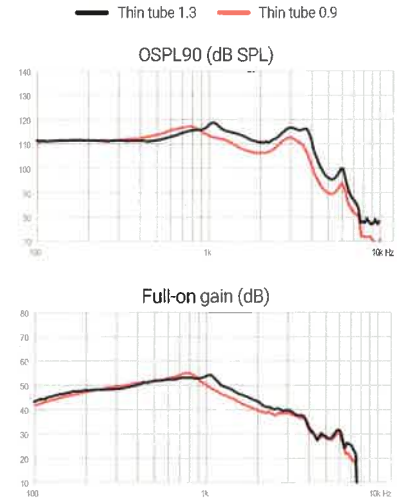
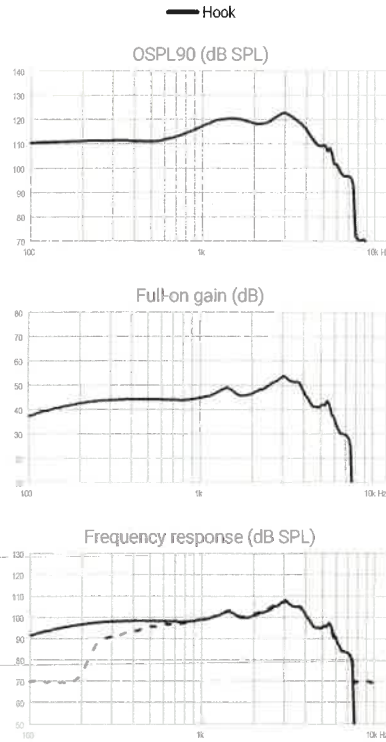
Measured according to ANSI S3.22: 2014,
IEC 60118-0:2015 and IEC 60318-5:2006.

Technical information
Omnidirectional mode is used
unless otherwise stated.

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- 1) Full-on gain is measured with the gain control of the hearing instruments set to its full-on position minus 20 dB and with an input SPL of 70 dB. This is to obtain a gain response equal to the full-on gain response from e.g. IEC 60118-0+A1:1994 but without influence of feedback.
- 2) Measurement is done in quiescent mode. Expected use time for rechargeable battery depends on use pattern, active feature set, hearing loss, sound environment, battery age and use of wireless accessories.

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 SBO Hearing A/S
Kongebakken 9
DK-2765 Smørum
Denmark

www.bernafon.com

Bernaфон is part of the Demant Group.